**poli
COLD**

CE 2460

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**poli
COLD**

CE 2460

POLICOLD is an acrylate for the fabrication of dentures, repairs and rebasings by using the cast technique and the cold curing.

Colour range: Pink, Pink Veined.

Indications:

- partial dentures
- repairs
- rebasings

Directions for use:

Visually inspect liquid. If you notice any sediment or gel in the liquid, discard it in accordance with local legislation.

1. Casting technique
 - Preparation
The model is to be isolated by an isolating fluid. A gypsum or silicon study model is to be elaborated.
The removal of wax is carried out according to common procedures.
Add liquid detergent to the boiling water in which the wax is washed out. The washing out with boiling water is to be continued until the fat is thoroughly removed from the teeth. The surface of the model is to be isolated by an isolating fluid. Apply the Policold fluid on the roughened teeth.
 - Mixing and casting
The recommended mixing ratio is 16g powder to 10g liquid (ratio of volumes 22ml powder to 11ml liquid). Place the liquid into a mixing jar and add the powder little by little. The liquid and the powder are to be mixed with a spatula for about 30 sec., then pour the liquid mixture into the prepared model and wait that the surface of the pouring material becomes rather dry. The model should be cold. The material is at room temperature 5 - 9 minutes after the pouring suitable for working. The working time depends on the temperature of the liquid and the powder before mixing as well as the surrounding room temperature.
 - Polymerisation
When the surface of the poured material isn't wet any more, the prepared model is to be put into the hot pressure vessel. The polymerisation conditions are the following: water temperature 40°- 45°C, pressure 6 bar, polymerisation time 20 minutes.
 - Trimming
The prosthesis is after the removal of the gypsum or the silicon trimmed and polished according to procedures common in dental laboratories.

2. Repairs and rebasings
 - Preparation
After the removal of wax from the prosthesis, apply the isolating fluid on the gypsum model. The parts of the prosthesis which are to be prepared are roughened and the Policold fluid is applied.
 - Mixing and casting
The recommended mixing ratio is 18g powder to 10 g liquid (ratio of volumes 24ml powder to 11ml liquid). Place the liquid into a mixing jar and add the powder

little by little. The liquid and the powder are to be mixed with a spatula for about 30 sec., after that the mixture is ready. At a room temperature of 22°C the working time is 5 - 9 minutes.

- Polymerisation
The prepared model is put into the hot pressure vessel. The conditions of polymerisation are the following: water temperature 40°- 45°C, pressure 6 bar, polymerisation time 20 minutes
- Trimming
After the removal of the gypsum, the prosthesis is trimmed and polished according to procedures common in dental laboratories

Storage:

Store at temperature 5-25°C. Avoid direct exposure to sun. Store in dry place.

Warning:

Close container immediately after use. Do not use material after expiry date. Keep out of reach of children.
Avoid direct contact between skin and uncured material as well as inhalation of the liquid.
In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of severely immunocompromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. The final use in this special group of patients should be weighed by the dentist according to risk/benefits ratio for each patient.
In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

Warning of liquid component:

Liquid is highly flammable. Irritating to eyes, respiratory system and skin.
May cause sensitization by skin contact. Keep container in a well ventilated place.
Keep away from sources of ignition. NO smoking. Do not empty into drains.
Take precautionary measures against static discharges.

Packaging:

Policold powder – 700g
Policold liquid – 500g
Policold Complet: powder – 200g
liquid – 150g

**We recommend the use with Polident Ref-line, Primodent or Cross-linked 2 teeth*

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