

EN The PRIMODENT teeth collection comprises 45 sets of teeth manufactured in a technological process which guarantees excellent mechanical properties and the colour resistance of the teeth. The collection consists of 27 sets of upper anterior, 10 sets of lower anterior, and 8 sets of posterior teeth. In terms of their mould the upper anterior teeth are divided into a group of triangular, a group of oval, and a group of square teeth. The available wide range of moulds meets even the most demanding criteria. The endosed Table facilitates the selection of compatible anterior and posterior teeth, the shade guides Primodent and Primodent - V, however, the choice of adequate shades.

** In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; vigilanca@polident.si) and the competent authority of the Member State, where the user is established.*

* En el caso de que se produzca algún incidente grave relacionado con el dispositivo, deberá informarse de inmediato al fabricante (Polident d.o.o.; vigilancia@polident.si) y a las autoridades competentes del Estado miembro en el que resida el usuario.

** Falls ein schwerwiegendes Vorkommnis im Zusammenhang mit dem Produkt aufgetreten ist, sollte dies dem Hersteller (Polident d.o.o.; vigilanca@polident.si) und der zuständigen Behörde des Mitgliedstaates, in dem der Anwender ansässig ist, gemeldet werden.*

** Sve ozbiljne štetne događaje koji se javе vezano uz ovaj proizvod treba prijaviti proizvođaču (Polident d.o.o.; vigilanca@polident.si) i nadležnom tijelu države članice u kojoj je korisnik poslovno registriran*

** In caso di gravi incidenti in relazione al dispositivo, questi devono essere riferiti al fabbricante (Polident d.o.o.; vigilanza@polident.si) e all'autorità competente dello Stato membro in cui ha sede l'utilizzatore.*

* Qualquer incidente grave relacionado com o dispositivo deve ser comunicado ao fabricante (Polident d.o.o.; vigilanca@polident.si) e à autoridade competente do Estado-Membro em que o utilizador está estabelecido.

* V primeru resnih težav v zvezi z uporabo izdelka naj se to sporoči proizvajalcu (Polident d.o.o., vigilanca@polident.si) in pristojnemu organu države članice, kjer ima uporabnik sedež.

SYMBOL, SIMBOLÓ, SYMBOL, SIMBOL, SIMBÓLO, SIMBOL, SIMBÓ	CE  	
EN EXPLANATION	EN CE marking -	EN Manufacturer
ES EXPLICACIÓN	With Notified body number	ES Fabricante
DE ERLÄUTERUNG	ES Marcado CE -	DE Hersteller
HR OBRAZLOŽENJE	com número de cuerpo notificado	HR Proizvođač
PT EXPLICAÇÃO	DE CE-Kennzeichnung -	PT Fabricante
IT ESPLICAZIONE	mit Nr. der benannten Stelle	IT Proizvajalec
SI POMEN	HR CE oznaka -	SI Proizvajalec
	s brojem prijavljene tijela	
REF	IT Marcatura CE -	MD
	com número de ente notificado	
EN Catalogue number	PT Marcação CE -	EN Medical device
ES Número de catálogo	com o número do corpo notificado	ES Dispositivo médico
DE Katalognummer	SI Oznaka CE -	DE Medizinisches Gerät
HR Kataložni broj	s številko prijavljene organa	HR Medicinski uređaj
IT Numero di catalogo		IT Dispositivo medico
SI Kataložna številka		PT Aparelho médico
		SI Medicinski pripomoček
LOT	Rx Only	
EN Batch code	EN Prescription only	EN Use - by date
ES Código de lote	ES Solo receta	ES Utilizar por fecha
DE Loschiffcode	DE Verschreibungspflichtig	DE Halbfabrikatdatum
HR Batch kod	HR Samo na recept	HR Upotreba - do datuma
IT Codice lotto	IT Solo su prescrizione	IT Usare entro la data
PT Código de lote	PT Somente receita médica	PT Use - por data
SI Številka lote	SI Samo na recept	SI Rok uporabe



mould chart

www.polident.si

Date: 6.12.2022

 since 1958

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

BARVE / BOJE / COLOURS / FARBE / COLOR / COR / COLORE
BL3
A1 - A2 - A3 - A3,5 - A4
B1 - B2 - B3 - B4
C1 - C2 - C3 - C4
D2 - D3 - D4
Y0 - Y1 -Y2 - Y3 - Y4 - Y5 - Y6
B1 - B2 - B3 - B4 - B5 - B6
G1 - G2 - G3 - G4 - G5 - G6

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

3	31	II	T1, S1
	32	I2, I3, L2	T1, T2, S1
	33	II, L2, I3	T2, T3
	34	I3, I4	T2, T3, T4
	35	I2, I4, I5, L2	T3, T4
	36	I4, I5, L6	T3, T4
	37	I5, I6	T4, T6
	37A	I6	T4, T6
	38	I6, I8, I7	T4, T6, S5
	39	I7	T6
0	32I	I2, I3, L2	T1, T2, S1
	34I	I3, I4	T2, T3, T4
	37I	I6	T4, T6
	01	II	T1, S1
	02	II, I3, L2	T1, T2, T3, S1
	03	I3, I4	T3, T4, S1
4	04	I4, I5, L6	T3, S5
	05	I5, I6, L8	T4, S5
	05A	L8, I6	T4, S5, T6
	06	I6, I7, L8	S5, T6
	41	II	T1, S1
	42	I3, I4, L2	T2, T3, S3
	43	I3, I4, L2	T3, T4
	43A	I3, I4, L6	T3, S3, S5
	44	I5, I6, L6	T4, S5
	45	I5, I6	T4, S5
	46	L8	T4, S5, T6

CERVICAL WIDER SHAPES ALSO
SUITABLE FOR IMPLANTS

primodent®
For beautiful smile.