# poli TEMP

CE 2460

ENG

**POLITEMP** is a cold-curing acrylic resin for temporary crowns and bridges.

**Colour range:** A1, A2, A3, A3.5, B3, C2, D3, Incisal

#### **Indications:**

- temporary crowns
- temporary bridges (3 to 4 parts)

#### **Directions for use:**

##### • **Preparation**

###### *Temporary crown*

Before the preparation of the cores in the mouth, a silicone impression is taken.

After the preparation of the core a new impression which is used for the casting of a class III stone model (Polidur) is prepared. In case of the preparation of the core under the gingiva an etching of the latter on the model is recommended, however, a free access to the preparation margin has to be assured. A wax-up has to be modelled and after that a new silicone impression to be taken in case no impression has been prepared prior to the core preparation. After the removal of the wax the stone model has to be insulated by an isolating fluid.

###### *Temporary bridge*

After the elaboration of the silicone impression a stone model is being casted which is then used for the elaboration of a temporary bridge. After the preparation of the stone model a wax-up has to be elaborated and then silicone impressed. The wax has to be removed and the model insulated by an isolating fluid.

##### • **Mixing and polymerisation**

The powder and liquid are mixed according to a volume ratio of 2,5 : 1. Pour the liquid into a mixing jar and add the powder little by little. The liquid and the powder are to be mixed with a spatula for approx. 30 sec. Cover the mixture in the mixing jar and put it at room temperature of 22°C for approx. 7 minutes aside. The consistency of the mixture after that period of time is suitable for treatment. The working time of the mixture at a room temperature of 22°C is 6 minutes. The working time depends on the temperatures of the liquid and the powder before the mixing as well as on the room temperature. The mixture has to be mixed again thoroughly and put into the impression. All together is then placed onto the model. After the application of the material into the impression and after that on the model you should wait for 2-3 minutes in order to enable the desired modelling of the material.

The material surplus is removed. The prepared model is placed into the pressure vessel. The polymerisation has to take place at the water temperature of 55-60°C, the pressure of 6 bar and in the polymerisation time of 10 minutes.

The product after the polymerisation of the dentine is being roughly treated and then enough space for the application of the incisal colour is being assured. The

dentine is to be covered by the liquid Politemp, the incisal colour has to be applied with the help of a modelling spatula, shaped and then once again polymerised.

##### • **Trimming**

The temporary crown or bridge after the removal of the stone and the silicone is being treated and polished according to procedures common in dental laboratories.

#### **Storage:**

Store liquid at temperature up to 25°C; store powder at temperature up to 35°C.

Avoid direct exposure to sun.

Expiry date shown on the packaging.

#### **Warning:**

In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. In case of severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient.

Close container immediately after use. Do not use material after expiry date. Keep out of reach of children.

Avoid direct contact between skin and uncured material as well as inhalation of the liquid.

In case that any serious incident that has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; [vigilanca@polident.si](mailto:vigilanca@polident.si)) and the competent authority of the Member State, where the user is established.

#### **Warning of liquid component:**

Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact. Keep container in a well ventilated place. Keep away from sources of ignition. NO smoking. Do not empty into drains.

Take precautionary measures against static discharges.

#### **Packaging:**

Powder and liquid are sold as two different products but there are always used together.

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