uni TRAY

CE

UNITRAY LC is a light curing tray material in form of prefabricated, upper baseplates to make rigid, custom impression trays and bite-blocks, as well as for other indications (refer to »indications«). UNITRAY LC can be cured in all ultraviolet and Halogen light curing units. The baseplates are supplied in a re-usable light-proof box which can be used for transporting or storing models etc.

Indications:

- Custom impression tray
- Bite-block
- Baseplate for setting up dentures
- Articulator guide pin excursion tracing
- Bite water for trying in framework
- Splinting outer coping or telescoping crowns

Preparation:

- Mark the outline of the future tray on the model.
- Block out the undercuts on the model.
- To create a spacer for the impression material, block out the entire model (e.g. with a sheet of wax) and cut stops out.
- If the plaster is very dry, soak the model briefly with water or apply an alginate isolation.

Application:

- Remove the baseplate from the box and close it.
- Once the model has been blocked out, adapt the baseplate to it and cut to shape using, e.g. a scalpel.
- Use the remaining material to form a handle and attach it to the tray.
- Place the model and tray in a light curing unit and cure without vacuum.
- Remove the tray from the model and polymerize its underside.
- If the tray has been cured in air, use alcohol to remove the smear layer.

Times:

Working at daylight 20 minutes

Polymerisation in an ultraviolet or Halogen light curing unit, depending on the unit in use:

1. The top of the tray, on the model
3-5 minutes
2. The underside of the tray, off the model
3-5 minutes

Trimming:

- Always use a dust extractor when trimming the tray!
- Trim the custom tray with tungsten carbide burs and smooth it with sandpaper. Pumice slurry can be used for polishing.

Caution:

Avoid any contact between the eyes and skin and the uncured material, i.e. also with the inhibited layer. The product has been developed for use in accordance with the usage directions. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore, the user is obliged under his own responsibility to examine the material for its suitability and its usage possibilities for the intended applications, especially if the latter are not detailed in the usage directions.

Hygiene:

Immerse impressions taken with UNITRAY LC. Customs trays in a disinfectant solution based on glutar aldehyde (the time depends on the disinfectant manufacturer's directions).

Incompatibility:

UNITRAY LC may cause sensitization in sensitive persons. Against the containings of UNITRAY LC (UNITRAY LC Contains acrylates). If such reactions are experienced, discontinue the use of UNITRAY LC.

Technical Data:

Flexural strength min. 130 N/mm²

Storage:

Do not store UNITRAY LC above 22°C
Protect against light.

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