CE 2460

POLIREPAR S

POLIREPAR S je akrilat za izdelavo reparatur s hladno polimerizacijo.

Barva: Pink

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Navodila za delo:

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CE 2460

POLIREPAR S

POLIREPAR S is an acrylate for the fabrication of repairs by using the cold curing.

Colour range: Pink

Indications:

- repairs

Directions for use:

- Preparation
After the removal of wax from the prosthesis, apply the isolating fluid on the gypsum model. The parts of the prosthesis which are to be prepared are roughened and the Polirepar S fluid is applied.
- Mixing and casting
The recommended mixing volumes ratio powder-liquid is 1,7-2,0 : 1. Place the liquid into a mixing jar and add the powder little by little. The liquid and the powder are to be mixed with a spatula for about 30 sec. Cover the mixture in the mixing jar and put it approximately 2 minutes aside. After that the mixture is ready. At a room temperature of 22°C the working time is 4 minutes.
- Polymerisation
The prepared model is put into the hot pressure vessel. The conditions of polymerisation are the following: water temperature 40-45°C, pressure 6 bar, polymerisation time 20 minutes.
- Trimming
After the removal of the gypsum, the prosthesis is trimmed and polished according to procedures common in dental laboratories.

Storage:

Store at room temperature up to 25°C. Avoid direct exposure to sun

Warning:

Close container immediately after use. Do not use material after expiry date. Keep out of reach of children.
Avoid direct contact between skin and uncured material as well as inhalation of the liquid.
In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of

severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. The final use in this special group of patients should be weighed by the dentist according to risk/benefits ratio for each patient.

In case that any serious incident that has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

Warning of liquid component:

Liquid is highly flammable. Irritating to eyes, respiratory system and skin.
May cause sensitization by skin contact.
Keep container in a well ventilated place.
Keep away from sources of ignition. NO smoking.

Do not empty into drains.

Take precautionary measures against static discharges.

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