poli HOT

POLIHOT je akrilat za izdelavo totalnih in parcijalnih protez s tehniko vroče polimerizacije.

Barve: Pink, Pink Veined

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- totalna proteza
- parcialna proteza

Navodila za delo:

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POLIHOT is an acrylate for the fabrication of full and partial dentures by using the heat curing.

Colours: Pink, Pink Veined

Indications:

- full dentures
- partial dentures

Directions for use:

Visually inspect liquid. If you notice any sediment or gel in the liquid, discard it in accordance with local legislation.

- **Preparation**
After the complete washing out of the wax from the flask, the surface of model is to be isolated by an isolating fluid and thoroughly dried. In order to ensure a tight attachment to the base plate of the prosthesis apply the Polihot liquid on the roughened teeth.
- **Mixing and packing/pressing**
The recommended mixing ratio is 22g powder: 10g liquid (30ml powder : 11ml liquid). Place the liquid into a mixing jar and add the powder little by little. The paste has to be thoroughly mixed and then put aside in a covered mixing jar for **3 minutes** at room temperature of 22°C. After that the paste has to be knead well and placed into a flask which is to be put into the pressure vessel.
WARNING: The paste has to be pressed into the flask within **7 minutes** after the liquid and powder have come into contact, otherwise the paste results in being too hard for pressing.
The dough time as well as the working time is subject to the room temperature as well as the temperature of the liquid and the powder before mixing.
- **Polymerisation**
Place the flask in the mounting device into a water bath filled with cold water, heat the water to 100°C and polymerize for 35 - 45 minutes.
- **Cooling**
Cool the flask down to room temperature.
- **Trimming**
Open the cooled down flask and remove the stone. The prosthesis is trimmed and polished according to procedures common in dental laboratories
- **Repairs**
The use of the cold curing material Policold is recommended, the colours of which match the

acrylate for the fabrication of prosthesis by using the heat curing (Polihot).

Storage:

Store at room temperature 5-25°C. Avoid direct exposure to sun. Store in dry place.

Warning:

Close container immediately after use. Do not use material after expiry date. Keep out of reach of children.

Avoid direct contact between skin and uncured material as well as inhalation of the liquid. In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. The final use in this special group of patients should be weighed by the dentist according to risk/benefits ratio for each patient.

In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.: vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

Warning of liquid component:

Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact. Keep container in a well ventilated place. Keep away from sources of ignition. NO smoking. Do not empty into drains. Take precautionary measures against static discharges

Packaging:

Polihot powder - 700g
Polihot liquid - 500g,

**We recommend the use with Polident Ref-line, Primodent or Cross-linked 2 teeth.*

Polident d.o.o. Slovenija, Volčja Draga 42,
5293 Volčja Draga, Slovenija
T: +386 53304840, F: +386 53304870
E-mail: polident@polident.si