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**ORTO**  
*poli*

CE 2460

Free of DMPTA means pleasant smell of the liquid and user friendlier handling.

PLEASE CHECK THE CONDITIONS OF THE POLYMERIZATION.

ORTOPOLI is an acrylate for the fabrication of orthodontic appliances by using the spray-on technique and the cold curing.

**Colour range:** red, blue, yellow, colourless, fl. yellow, fl. orange, pink, green, indigo blue

**Indications:** Orthodontic appliance

**Directions for use:**

- Preparation  
Attach the wires onto the model and soak it in water. Afterwards it has to be dried thoroughly.
- Spray/sprinkle technique  
Wet the surface of the model with the help of a dripping-tube. Shake the powder thoroughly before use. With the help of a shaker apply a thin layer of powder on the model. Wet the powder with liquid. Continue applying alternating thin layers to the model until the desired thickness of the plate has been achieved.
- Polymerisation  
After the spray/sprinkle procedure is finished, set the prepared model into the pressure vessel. The conditions of polymerisation are the following: water temperature 60°C, pressure 6 bar, polymerisation time 20 minutes.
- Trimming  
The orthodontic appliances, after the polymerisation, can be trimmed if necessary according to procedures common in dental laboratories.
- Repairs  
Repairs are carried out with the use of Ortopoli. Add powder to the liquid in a ratio of 10 g powder : 6 ml liquid (ratio of volumes 2.5 : 1). Mix the powder and the liquid and apply the mixture in a thin layer on the damaged part of the orthodontic appliance. Continue applying alternating layers of powder and liquid as described beforehand, until the desired thickness of the plate has been achieved. The polymerisation is carried out according to

the already described procedure.

**Storage:**

Store at temperature 5-25°C. Avoid direct exposure to sun. Store in dry place.

**Warning:**

Close container immediately after use.  
Do not use material after expiry date.  
Keep out of reach of children.  
Avoid direct contact between skin and uncured material as well as inhalation of the liquid.  
In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. The final use in this special group of patients should be weighed by the dentist according to risk/benefits ratio for each patient.

In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; [vigilanca@polident.si](mailto:vigilanca@polident.si)) and the competent authority of the Member State, where the user is established.

**Warning of liquid component:**

Liquid is highly flammable. Irritating to eyes, respiratory system and skin  
May cause sensitization by skin contact  
Keep container in a well ventilated place.  
Keep away from sources of ignition. NO smoking.  
Do not empty into drains.

Take precautionary measures against static discharges.

**Packaging:**

Ortopoli powder, colourless – 700g, dosing dispenser  
Ortopoli liquid, coloured or colourless – 500g  
Ortopoli liquid, coloured 150g (fl. yellow, fl. orange, indigo blue, green, pink).

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