EN

K-30 S is an acrylate for the fabrication of full and partial dentures by using the packing/pressing technique and the classical heat curing.

Color range: P, PV, PN, PN-V, M26, M26V

Indications: full dentures, partial dentures

DIRECTIONS FOR USE:

➤ Preparation: The flasking and the removal of wax from the flask is carried out according to procedures common in dental laboratories. Add a liquid detergent to the boiling water in which the wax is washed out. The washing out with boiling water is to be continued until the fat is thoroughly removed from the teeth. The surface of the model is to be isolated by an isolating fluid and thoroughly dried. In order to ensure a tight attachment to the base plate of the prosthesis, apply the K-30 S fluid on the roughened teeth.

➤ Mixing and packing/pressing: The recommended mixing ratio is 23 g powder to 10 g liquid (volumen ratio 3:1). Place the liquid into a mixing jar and add the powder little by little. The liquid and the powder are to be mixed with a spatula for about 30 sec. Cover the mixture in the mixing jar and put it at room temperature of 22°C approximately 30 minutes aside. The consistency of the mixture after that is suitable for the packing (the mixture doesn't stick anymore and separates cleanly from the sides of the mixing jar).The working time of the mixture is of approximately 15 minutes. The dough time as well as the working time is subject to the temperature of the liquid and the powder before mixing and the room temperature. Knead the mixture with your fingers and place it into the flask. The use of a cold flask is recommended. Close the flask carefully and put it into the pressure vessel. The pressure in the pressure vessel is to be increased gradually up to 80 bar and to be left thus for 10 minutes. Then the flask is put into a mounting device.

➤ **Polymerisation: You can choose among the following polymerisation types: 1. Place the flask in the mounting device into a water bath filled with cold water, heat the water to 100°C and polymerize for 35-45 minutes. Or 2. Place the flask in the mounting device into a water bath filled with cold water, heat the water in 30 minutes up to 70°C and polymerize for 30-45 minutes. After that the water is to be heated in 30 minutes up to 100°C. Polymerize for 30-45 minutes.**

➤ Cooling: Cool down the flask for 30 minutes in the open and after that in a cold water bath.

➤ Trimming: Open the cooled down flask and remove the model. The prosthesis is trimmed and polished according to procedures common in dental laboratories.

➤ Repairs: The use of the cold curing material Polirepar S is recommended, the colours of which matches the acrylate for the fabrication of prosthesis by using the heat curing (K-30 S).

Storage: Store at room temperature up to 25°C. Avoid direct exposure to sun.

Warning: Close container immediately after use. Do not use material after expiry date. Keep out of reach of children. Avoid direct contact between skin an uncured material as well as inhalation of the liquid. In case of known allergies or hypersensitivity to acrylic materials do not use the product. In case of severely immuno compromised patient, the doctor should decide and weigh the benefit/risk with regards to the individual patient. Due to lack of information regarding the safety of PMMA exposure for pregnant women, we advise to avoid using the device for pregnant women. The final use in this special group of patients should be weighed by the dentist according to risk/benefits ratio for each patient. In case that any serious incident that has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

Warning of liquid component: Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact. Keep container in a well ventilated place. Keep away from sources of ignition. NO smoking. Do not empty into drains. Take precautionary measures against static discharges.

Packaging: K-30 S powder 1000g, 700g, 350g and K-30 S liquid 500ml, 200ml

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