# Cross Linked 2

**EN** Cross Linked 2 teeth are highly cross linked polymethylmethacrylate teeth. High quality raw materials along with an adequate technological production procedure assure the colour and heat stability of the teeth, a high resistance to abrasion and solvents as well as a durable adhesion with the denture base material. Cross Linked 2 teeth consist of 18 different moulds of upper anterior teeth which are subdivided into 3 groups (the oval, square and triangular group), of 7 different moulds of lower anterior and of 6 different moulds of posterior teeth. Cross Linked 2 teeth are successfully fitted into partial and total prothesis as they perfectly replace the teeth of the patients.

*\* Roughen and wet the ridge lap surface of artificial teeth with dental monomer liquid to achieve better bonding to the denture base.*

*\* In case of known allergies or hypersensitivity to acrylic materials do not use the product.*

*\* In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; [vigilanca@polident.si](mailto:vigilanca@polident.si)) and the competent authority of the Member State, where the user is established.*

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