EN List of symbols  
CS Seznam symbolů  
DE Liste der Symbole  
ES Lista de símbolos  
FR Liste des symboles  
HR Popis simbola  
IT Lista dei simboli  
PT Lista de símbolos  
SK Zoznam symbolov  
SL Seznam simbola  
SV Symbolförklaring  
UK Список символов

EN Manufacturer  
CS Výrobce  
DE Hersteller  
ES Fabricante  
FR Fabricant  
HR Proizvođač  
IT Produttore  
PT Fabricante  
SK Výrobca  
SL Proizvajalec  
SV Tillverkare  
UK Výrobník

EN Liquid  
CS Tekutina  
DE Flüssigkeit  
ES Líquido  
FR Liquide  
HR Tekućina  
IT Liquido  
PT Líquido  
SK Kvapalina  
SL Tekočina  
SV Vätska  
UK Рідина

EN Powder  
CS Prášek  
DE Pulver  
ES Polvo  
FR Poudre  
HR Prášak  
IT Polvere  
PT Po  
SK Prások  
SL Praha  
SV Pulver  
UK Порошок

EN Colour  
CS Barva  
DE Farbe  
ES Color  
FR Couleur  
HR Boja  
IT Colore  
PT Cor  
SK Farba  
SL Barva  
SV Färg  
UK Konip

REF EN Catalogue number  
CS Katalogové číslo  
DE Katalognummer  
ES Número en catálogo  
FR Numéro de catalogue  
HR Kataloški broj  
IT Numero di catalogo  
PT Número do catálogo  
SK Katalógové číslo  
SL Kataloška številka  
SV Katalognummer  
UK Номер за каталогом

LOT EN Batch code  
CS Šáre  
DE Chargencode  
ES Código de lote  
FR Code du lot  
HR Šifra serije  
IT Codice del loto  
PT Código de lote  
SK Kod súrže  
SL Koda serije  
SV Satskod  
UK Kod parťat

UDI EN Unique device identification  
CS Jedinečná identifikace prostředku  
DE Eindeutige Geräteidentifikation  
ES Identificador único de dispositivo  
FR Identification unique de l'appareil  
HR Jedinstvena identifikacijska oznaka uređaja  
IT Identificazione univoca del dispositivo  
PT Identificação única do dispositivo  
SK Jedinečná identifikácia pomôcky  
SL Edinstveni identifikator pripomočka  
SV Unik produktidentifier  
UK Унікальна ідентифікація пристрою

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General description:  
POLITRAY is an autopolymerizable acrylic resin for the fabrication of individual impression tray of a patient's teeth and surrounding oral structures for further processing. It is designed to accurately capture the unique shape and contours of a patient's mouth to aid in various dental procedures, such as crowns, bridges, dentures, and orthodontic appliances.  
Material is supplied in form of liquid and powder that shall be always used together by a dental health care professional (dental technician).

Indications: individual impression tray

Composition:  
Powder: acrylic copolymer, polymethyl methacrylate, calcium carbonate, benzoyl peroxide and pigment  
Tekuina: methyl methacrylate, paraffin oil, aktivator

Contraindications:  
Do not use in case of known allergy/hypersensitivity to acrylics or any other component.

Safety data sheet available on the web page.  
In the event of a serious incident that has occurred in connection with the device, this should be reported to the manufacturer (Polident d.o.o.; [vigilancia@polident.si](mailto:vigilancia@polident.si)) and the competent authority of the Member State, where the user is established.

DIRECTIONS FOR USE

Visually inspect liquid. If you notice any sediment or gel do not use the liquid. Discard it in accordance with local legislation. We recommend temperature conditioning of the liquid and powder components at the room temperature before use.

- Preparation: Block out the undercuts of the plaster model with wax. Soak the plaster model in water and wait for the model's surface to dry before proceeding.

- Mixing and fabrication of the individual impression tray:  
Shake well the powder before use to get a homogenous material. The recommended mixing ratio is 30 g powder : 10 ml liquid (ratio of volumes 3:5:1). Pour the liquid into a mixing jar and gradually add the powder with a spatula for about 30 seconds. The consistency of the mixture is suitable for the preparation of the impression tray when the mixture doesn't stick anymore and separates easily from the sides of the mixing jar. The mixture can be handled for approximately 4.5 minutes at a room temperature of 22°C. The working time is subject to the surrounding room temperature. Ensure the handle is sufficient in size and positioned effectively for practical use.

- Polymerisation: At room temperature of 22°C the autopolymerisation of the individual impression tray mixture takes place in 30 minutes.

- Trimming: After the polymerisation the impression tray is removed from the model and trimmed according to procedures common in dental laboratories

Information for dentist:  
In accordance with standard dental medicine practices, dentist must clean medical device before inserting it into the oral cavity.

Storage:  
Store the liquid at temperatures up to 25 °C, keep protected from sunlight. Store powder at temperature up to 25°C, keep protected from sunlight, keep dry.

General warnings:  
Close container immediately after use. Do not use material after expiry date. Expiry date is printed on the packaging. Keep out of reach of children. Avoid direct contact between skin and uncured material as well as inhalation of the liquid. Do not store the powder at temperatures above 50°C for long period of time. Keep the liquid and powder in the original packaging only. Do not use if the primary packaging is damaged. It is necessary to wash your hands after handling the uncured material. Please ensure sufficient extraction of the particles from the workplace during the finishing steps. Wear suitable protective gloves. If no technical protective measures are taken, irritation of the eyes, skin and respiratory organs may occur when finishing the acrylics.

Warnings of liquid component:  
Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact. Bottles must be tightly closed and stored in well-ventilated place. Keep away from heat, sparks, open flames, and other sources of ignition. Do not smoke. Do not empty into drains.

Disposal:  
Disposal should be made in accordance with local, state and national legislation.

Packaging  
Politray powder 350g, 700g  
Politray liquid 150ml, 500ml

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