de

en

fr

es

it

nl

da

no

sv

fi

pt

cs

pl

uniTRAY LC

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uniTRAY LC

Instructions for use

Patient target group
All persons for whom individual trays are required for treatment.

Intended user
Dentist

Composition
Qualitative data:
Oligomeric urethane acrylate, vinyl ester resin, 2,2-Dimethoxy-1,2-diphenylethan-1-on ou 2,2-Dimethoxy-2-phenylacetophenone, (+)-2,3-Dioxy-1,7,7-dimethyl-bicyclo-(2,2,1)-heptane-2,3-dione, Diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide, 2-hydroxy-2-methyl-1-phenylpropane-1-on, 2-ethylhexyl-4-(dimethylamino)benzoate, 2,2-Diethoxyacetophenone, sintered glass, amorphous silica, fused pigment Blue 146 = CI 12485 / Pigment Blue 29 = CI 77007 / D&C Red 30 = CI 73300 / Pigment Blue 41 = CI 71601 / Pigment Violet 19 = CI 73900, Aromatics (Optamint).

Quantitative data:
100 g contains 19 g of acrylates/methacrylates and 80 g of verre.

Intended purpose of the product
Untray is a light-curing tray material in the form of pre-fabricated, baseslides to make rigid, custom impression trays and bite-blocks, as well as for other indications. Untray can be cured in all ultraviolet and LED light-curing units. The baseslide contains silica, do not use.

Indications
Custom impression tray

Contraindications
The material contains methacrylates.

Do not use in patients with an allergy to methacrylates or other substances contained in the product.

Performance characteristics of the product
Untray polymerises with light to form a solid and compact product.

Specific precautions
Avoid any contact between the eyes and skin and the untray material, i.e. also with the inhibited layer. Always use a dust extractor when trimming the tray!

Preparation/Manufacture
- Block the outline of the future tray on the model.
- Mark out the undercuts on the model.
- To create a spacer for the impression material, stop out the entire model (e.g. with a sheet of wax) and cut out the spacer.

- If the plaster is very dry, soak the model briefly in water or apply a wet isolation or an alginate isolation.
- Remove the baseslide from the box and clean it.
- Once the model has been blocked out, unblock and stop out.
- Use the remaining material to form a handle and attach it to the tray.
- Place the model and tray in a light-curing unit and cure without vacuum.

- Remove the tray from the model and polymerize once again from the underside.
- If the tray has been cured without a vacuum, use alcohol to remove the smear layer.
Work:
Time at daylight 20 minutes

Polymerisation in an ultraviolet or LED light-curing unit, depending on the unit's use:

Top of the tray, on the model 3-5 minutes
Underside of the tray, off the model 3-5 minutes

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Hygien:
Immerse impressions taken with Untray customs trays in a disinfectant solution based on glutaraldehyde. The time depends on the disinfectant manufacturer's directions.

The finished medical product is used by the dentist, e.g. for individual trays with impression material.

Caution
The product has been developed for use in the dental sector and must be used according to the directions. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore the user is obliged under his own responsibility to examine the material for its suitability and its use possibilities for the intended applications, especially if the latter are not detailed in the directions for use.

Adverse effects
Adverse side effects are not known.

Interactions
Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage
The Product should be stored between 0° C - 22° C

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The finished medical prod is used by the dentist, e.g. for individual trays with impression material.

Caution
The product was developed for use in the dental sector and must be used according to the directions. The manufacturer accepts no liability for damage that results from use other than the above. Furthermore the user is obliged under his own responsibility to examine the material for its suitability and its use possibilities for the intended applications, especially if the latter are not detailed in the directions for use.

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