ОртоПОЛ Cold-curing acrylic resin for orthodontic appliances by using the spray-on technique

General description:
Ortopol is an orthodontic base acrylic resin for the fabrication of orthodontic appliances by using the spray-on technique and cold curing technique.

Material is supplied in form of liquid and powder that shall be always used together by a dental health care professional (dental technician).

Material characteristic	Specification	Method
Sorption	< 32 µg/mm³	EN ISO 20795-2
Solubility	< 5 µg/mm³	EN ISO 20795-2
Residual monomer	< 5 %	EN ISO 20795-2
Maximum stress intensity factor	> 1,1 MPa m ^{1/2}	EN ISO 20795-2
Total fracture work	> 250 J/m²	EN ISO 20795-2

Variants: Available in various colours and quantities. Consult your distributor.

Indications: Malocclusion

Composition: Powder: polyethyl methacrylate, dibenzoyl peroxide Liquid: methyl methacrylate, ethylene glycol dimethacrylate, activator and pigments

Contraindications: Do not use in case of known allergy/hypersensitivity to acrylates or any other component. Safety data sheet available on the web page. *In the event of a serious incident that has occurred in connection with the device, this should be reported to the manufacturer (Polident d.o.o., vigilanca@polident.si) and the competent authority of the Member State, where the user is established.*

DIRECTIONS FOR USE:

Visually inspect liquid. If you notice any sediment or gel do not use the liquid. Discard it in accordance with local legislation. We recommend temperature conditioning of the liquid and powder components at the room temperature before use.

- Preparation:** Firmly attach the wires to the model and soak it in water. Following this, the material must be thoroughly dried.
- Sprays/prinkle technique:** Wet the model surface with the help of a dripping-tube. Shake the powder thoroughly before use. Apply a thin layer of powder on the model with the provided container. Wet the powder with liquid. Continue adding thin layers to the model until the desired plate thickness has been achieved.

- Polymerisation:** *After the sprays/prinkle procedure is finished, set the prepared model in the pressure vessel. Polymerisation conditions are: water temperature 60°C, pressure 6 bar, polymerisation time 20 minutes.*

- Trimming:** Following polymerisation, orthodontic appliances can be trimmed, if necessary, according to dental laboratory procedures.
- Repairs:** Repairs are done using Ortopol. Add powder to the liquid in a ratio of 10 g powder : 6 ml liquid (ratio of volumes 2.5 : 1). Mix the powder and the liquid and apply the mixture in a thin layer to the damaged part of the orthodontic appliance. Continue applying alternating layers of powder and liquid as described beforehand, until the desired plate thickness has been achieved. Polymerisation is carried out according to the previously described procedure.

NOTE! Dental restorations made of acrylics always contain a small proportion of residual monomer. The residual monomer content is below 5% if the polymerisation is performed according to the instructions for use. For further free monomer reduction it is recommended to store a medical device in water for at least 24 hours. If this step is not possible, advise the patient to place the appliance in water whenever it is not being worn.

Not following the instructions may cause residual monomer levels above safety limit, poor physical properties, or biofilm formation due to inadequate surface quality.

Information to be provided to the dentist:

In accordance with standard dental medical practices, dentist must clean medical device before inserting it into the oral cavity. The dentist must provide the patient with instructions for adequate cleaning of the medical device and follow up the patient at regular appointments to assess the condition of the medical device. We recommend cleaning with soft brush and non-abrasive toothpaste. Non-compliance with these instructions may result in increased biofilm formation or discoloration of the material.

Storage: Store the liquid at temperatures up to 25 °C, keep protected from sunlight. Store powder at temperature up to 30°C, keep protected from sunlight, keep dry.

General warnings for the dental technician: Do not use material after expiry date. Expiry date is printed on the packaging. Keep out of reach of children. Keep the liquid and powder in the original packaging only. Do not use if the primary packaging is damaged. Avoid direct contact between skin and uncured material as well as inhalation of the vapours. It is necessary to wash your hands after handling the uncured material. Close containers immediately after use. Ensure efficient ventilation of the workplace. Ensure sufficient extraction of the particles from the workplace during the finishing steps (trimming, grinding, polishing). Wear suitable personal protective equipment (PPE) during all procedures. If no technical protective measures are taken, irritation of the eyes, skin and respiratory organs may occur during the handling with uncured material and finishing procedures. Consult the Safety Data Sheet (SDS) for proper personal protective equipment (available on the website https://www.polident.si/). Do not store the powder at temperatures above 50°C for long period of time.

Warnings for liquid component: Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact (allergic skin reaction). Keep away from heat, sparks, open flames, and other sources of ignition. Do not smoke. Avoid breathing vapours. Bottles must be tightly closed and stored in well-ventilated place. Do not get in eyes, on skin, or on clothing. Wear protective gloves/protective clothing/eye protection. Take off immediately all contaminated clothing. Rinse skin with water or shower. Do not empty into drains. Take precautionary measures against static discharges.

Disposal: Do not empty unused product in drainage systems! The product must not be disposed of with municipal waste. Dispose in accordance with the local and/or national regulations.

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