EN INSTRUCTIONS FOR USE

INTENDED PURPOSE: Zirlux Transparent PMMA disc is suitable for milling removable occlusal splints.

COMPOSITION: PMMA material with embedded pigments and residual methyl methacrylate less than 1%.

INDICATION: Temporomandibular joint disorders and teeth protection

CONTRAINDICATIONS: Do not use the product in case of known allergies or hypersensitivity to acrylic materials.

WEARING TIME: Occlusal splints are recommended for use up to 1 year.

NOTE! Before use of the disc carefully read the label of the disc to choose the correct height of the disc.

RULES TO CONSIDER WITHIN THE DESIGN OF SPLINT: Splints can be individualized by adding light-curing or cold-curing dental acrylic materials. Thickness and cross-sections should be taken into consideration.

FINISHING STEP: Tools for acrylate are used for finishing and final adjustments. For polishing use pumice stone, pastes and soft polishing brushes to achieve a glossy surface.

CLEANING: Make sure to thoroughly clean the restoration to remove any residue of the milling and finishing processes using common dental laboratory practice (e.g. steam jet or mild soap/detergent).

INFORMATION FOR DENTIST: In accordance with standard dental medicine practices, dentist must clean medical device before inserting it into the oral cavity. The dentist must provide the patient with instructions for adequate cleaning of the medical device and follow up the patient at regular appointments to assess the condition of the medical device. We recommend cleaning with soft brushes and mild soap for removable medical device.

STORAGE: Keep away from direct sunlight!

DISPOSAL: Disposal should be made in accordance with local, state and national legislation.

WARNINGS: Professional use only! Dental CAD/CAM skills are required to properly process the product. The discs must be milled using a compatible CAD/CAM system. It is important not to overheat the material! Do not use the material after expiry date. Expiry date is printed on the packaging. PMMA dust may develop during the milling process, leading to mechanical irritation of the eyes, skin, and respiratory tract. Make sure to use the extraction system in the workplace. When processing PMMA discs, wear personal protective equipment (protection mask for dust, safety glasses, ...). To ensure traceability and assignment of the product data as well as the LOT numbers for the patient at any time, it is recommended to store the milling blank in the original packaging during two milling operations.

Further safety-relevant information can be found in the material safety data sheet available on request.

In case that any serious incident has occurred in relation to the device it should be reported to the manufacturer (Polident d.o.o.; vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

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HENRY SCHEIN SERVICES GmbH